



DENTAL PLUS

# HIGH PERFORMANCE POLYMERE

FOR CAD/CAM AND  
INJECTION MOLDING SYSTEM

INSTRUCTION MANUAL

ZAHN • HEIL • KUNST • STOFFE



## THE BIG DIFFERENCE



Different production processes decide if carbon becomes coal or a diamant.

modern high performance polymere replace in the high technology like space, aircraft and vehicle manufacturing more and more the goup of metal.  
High performance polymere gain in the dental technologie a higher ästhetik und a better comfort.

DentalPlus GmbH

Kohlgrub 5  
83122 Samerberg • Germany  
Telefon: +49 (0) 8032 / 989 20 07

Mo – Do von 09.00 –17.00 Uhr  
Fr von 9.00 –14.00 Uhr

Fax: +49 (0) 8032/ 988 27 90

eMail: [info@dentalplus.info](mailto:info@dentalplus.info)

## HIGH TECH IN DENTAL TECHNOLOGY.

Plastics technology has produced numerous Developments in recent years. Thereby the modern high-performance plastics also increasingly introduced into the world of medicine and especially dentistry.

Economic factors and problem awareness at Patients with respect to the mixing of metals in the mouth, support this change and the use of advanced plastics.

Most commonly so called PMMA plastics are applied in the dentistry and technology. This is a Hydrocarbon compound, which is available in a wide variety of quality levels.

Our expertise lies in the production of the material blanks. The technology, we have developed is advanced. The result is the special quality.

There are two basic methods of production of the blanks:

The **chemoplastic**, wherein a large plastic plate is shaped with powder and monomer. Similar to the powder-liquid method in dentistry.

The **thermoplastic**, wherein a granulate is plasticized by heat and is injected under high pressure into a mold.

**Thermoplastic** produced shaped bodies such as blanks, are characterized by extremely high homogeneity. By the high surface density these plastics are much more fracture stable (chemoplastic blanks at approx 1800-2400 MpA and thermoplastic blanks at approx 3000-3600 MpA).

Due to this high stability, it is possible, to produce plastic crowns and bridges for the permanent using (> 6 years).

In contrast to the chemoplastics, who show a higher content of residual monomers and may result in allergic or Health impairing reactions, thermoplastics haven't these problems due to the manufacturing process.

Modification of plastics by color pigments make the finishing easier and by fillers, such as glass, optimized thermoplastics (composites) meet the requirements for abrasion stability and surface hardness.

By CAD-CAM technology facilitate our specially designed plastic the work of a dental technician and provides special opportunities in patient care.

Our range offers numerous possibilities. Let us surprise you.


Learn more about plastics by visiting our website at: [www.dentalplus.info](http://www.dentalplus.info)



### Thermoplastic (high homogeneity)

E-module 3.000 - 3.600 MpA

- fracture stable
- plaque resistant
- dimensionally stable (> 6 years)

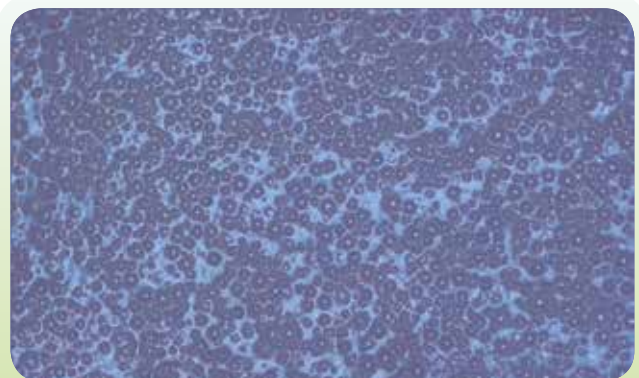

### Chemoplastic (inhomogeneous structure)

E-module 1.800 - 2.400 MpA

- low fracture stability
- plaque susceptible
- premature aging due to higher water



# Polyan Plus<sup>®</sup>

(highly resistant to breakage)

## chemical name

high performance-PMMA  
Medical-grade

## application area

- crowns and bridges
- model casting
- splint technique
- fullanatomic longterm interim prothetic
- framework
- total-/part denture

## advantages

- abrasionsstabel
- extreme fracture stability
- plaque resistant und biokompatibel
- colorconstancy and high translucent
- high efficiency
- high flexural strength

## worth knowing

**Prevent contact with alcohol (60-90%)  
over 3 min! Otherwise rist of craquelling  
crack.**

rebasng, repairing, extension with cold  
polymerisaten possible, thermopastic  
repairing possible

## specifications

elasticity modulus	3370 MPa
tensile strength	76,3 MPa
breaking tension	70,3 MPa
flexural strength	120 MPa

## handling injection molding

preheating cartouches	15 Min.
injection temperature	260°C
injection pressure	9 - 9,5 bar
holding pressure	1 Min.

## indications

### flexibility



### chemical compound



### polishing



### plaqueresistance



### translucency



applications:





colors injection molding systems

clear • pink • EU65 • pink opal • pink veined

colors CAC/CAM

clear • pink opal • pink2

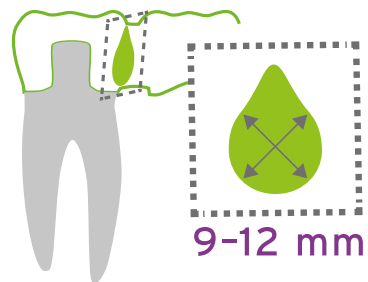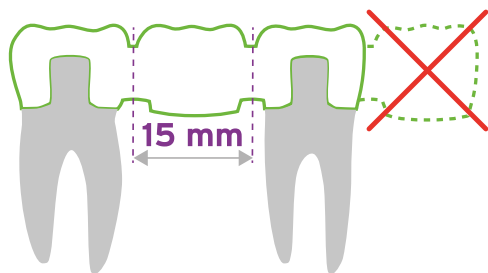





applications:





colors injection molding systems

A • A3,5 • B • B1 • B4 • C • C4

colors CAC/CAM

Monoblanc: A1 • A2 • A3 • A3,5 • B1 • B2 • B3 • C1 • C2 • C3 • bleach Multicolor: A1-A2 • A2-A3 • S1-A3,5 • A3-A3,5 • A2-A3 pink opal • B1-B2 • B2-B3 • B3-B4 • C1-C2 • C2-C3

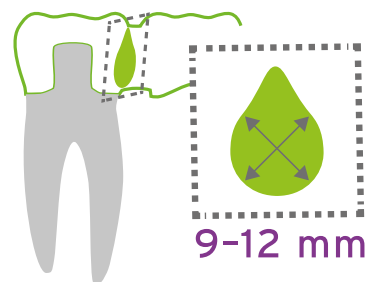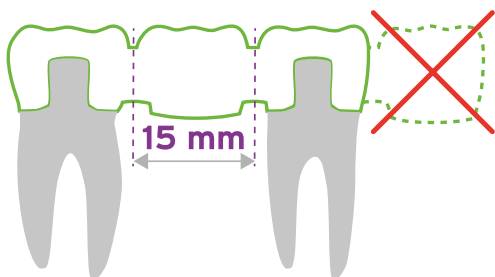





applications:





colors injection molding systems

A1 • A2 • A3 • A3,5 • pink • pink opal • pink2 • clear

colors CAC/CAM

Mono: A1 • A2 • A3 • A3,5 • pink2 • pink opal  
clear • B1 • B2 • B3 Multicolor: A2-A3,5  
• A2-A3 • A1-A2 • B1-B2 • B2-B3 • C2-C3

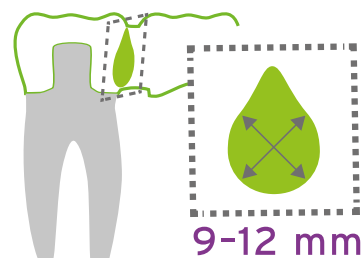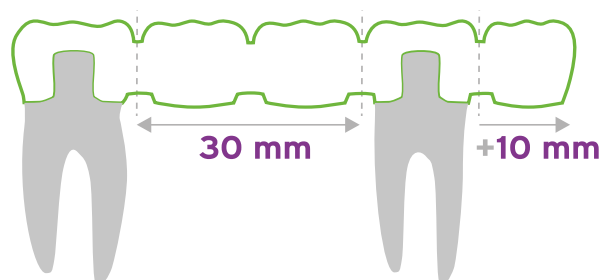



# Flexistrong Plus<sup>®</sup>

(unbreakable \*)

## chemical name

high performance-PVDF  
Medical-grade

## application area

- crowns and bridges
- model casting
- splint technique
- fullanatomic longterm interim prothetic
- framework
- total-/part denture

## advantages

- extreme fracture stability
- plaque resistant
- high translucent
- high chemical bond with other plastics
- veneering

## worth knowing

Very high ästhetic translucenc. resistance to plaque depend on the patients care.

extreme fracture stability, high toughness, extrem less water absorptio.

### \*almost unbreakable

Can be destroyed by improper handling.

## specifications

elasticity modulus	2000 MPa
tensile strength	90 MPa
ball indentation hardness	80 MPa
breaking tension	145°C
water absorption	<0,04%

## handling injection molding

preheating cartouches	15 Min.
injection temperature	220°C
injection pressure	6 -7 bar
holding pressure	1 Min.

## indications

### flexibility



### chemical compound



### polishing



### plaqueresistance



### translucency



applications:





colors injection molding systems

nature • pink • pink opal • A2 • A3,5

colors CAC/CAM

nature • pink • A2 • A3,5

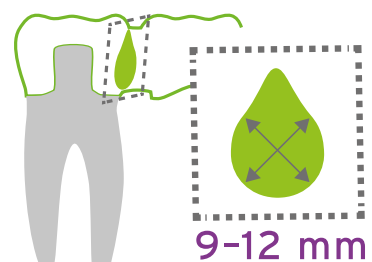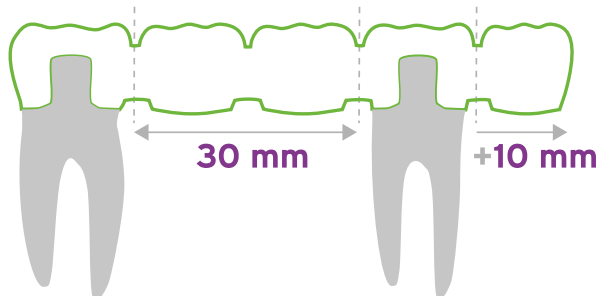





applications:



colors injection molding systems

nature • pink • A

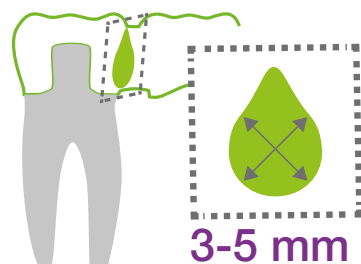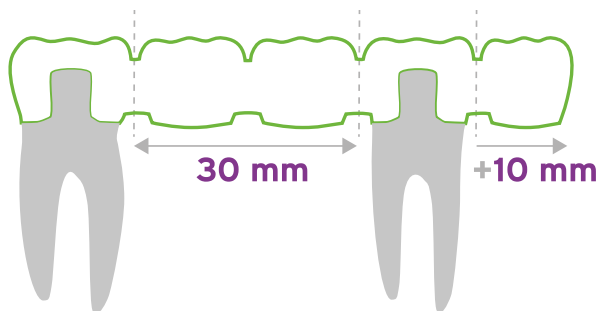





# PEEK

(very high resistance to breakage)

## chemical name

High performance polymer of the PEA group  
Medical-grade

## application area

- crowns and bridges
- model casting
- splint technique
- fullanatomic longterm interim prothetic
- framework
- total-/part denture

## advantages

- abrasionsstabel
- extreme fracture stability
- high flexural strength
- plaque resistant and biocompatible
- colorconsistency and high translucent
- high efficiency

## worth knowing

The absence of transluzenz limit the application of PEEK. With an suitable sandwich-technique you can get with this material an extrem high fracture stability with high ästhetic.

Friktive elemets for telescop or bar technique find in this polymer a perfect realisation.

## specifications

elasticity modulus	4000 MPa
tensile strength	180 MPa
ball hardness	230 MPa
water absorption	0,2 %

## handling injection molding

preheating cartouches	15 Min.
injection temperature	400°C
injection pressure	9 bar
holding pressure	1 Min.

## indications

### flexibility



### chemical compound



### polishing



### plaqueresistance



### translucency



applications:





colors injection molding systems

A • pink

colors CAC/CAM

nature (light grey)

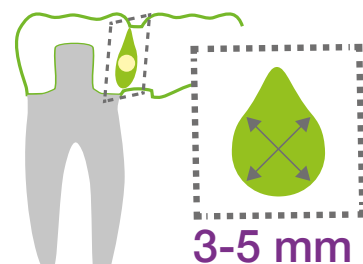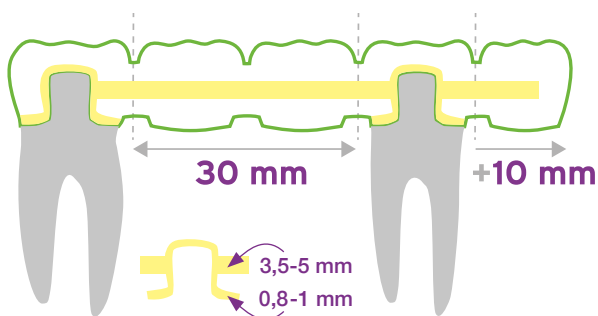



applications:





colors injection molding systems

nature • pink • B

colors CAC/CAM

no blancs

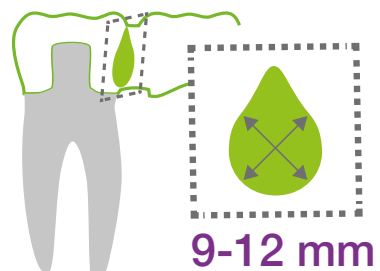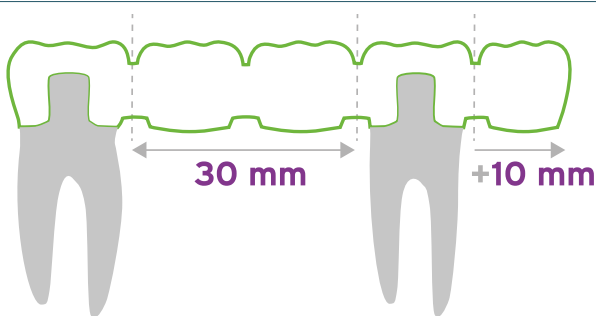



DENTAL PLUS

We effort to advise and inform you  
very comprehensive.

Distributor:

DentalPlus GmbH

Kohlgrub 5 • 83122 Samerberg • Germany

Telefon: +49 (0) 8032 / 989 20 07

Fax: +49 (0) 8032/ 988 27 90

eMail: [info@dentalplus.info](mailto:info@dentalplus.info)

Mo – Do 09.00 –17.00 Uhr

Fr 9.00 –14.00 Uhr

Delivery ex works plus transport and packing.

© 3. Edition 2021

We are grateful for the extensive range of footage the has been made  
available to us by very innovative colleagues.

All rights reserved. No part of the work may be produced in any form  
(fotography, microfilm or others) Without the written permission of the author  
or processed, dublicated or dishibuted using electronic systems.

