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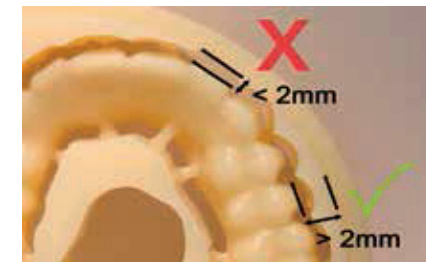
2. Characteristics/properties of **Eldy Plus®** Blanks

- extreme fracture stability
- high translucency
- can be veneered
- high chemical bonding to other resins possible

3. Composition

Special plastic material based on PETG

4. Material-mechanical specifications

| | | |
|-------------------|-------------|---|
| E- Modul | Ø 1.588 MPa | DIN EN ISO 20795-1:2013 compliant |
| Tensile strength | Ø 44,9 MPA | |
| Fracture stress | - | |
| Flexural strength | Ø 66,4 MPa | |

5. Delivery forms of the Blanks

[See current CAD/CAM catalog](#)

6. Indication

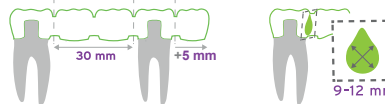
The material is suitable for producing

- Splints
- Partial and full dentures
- Crowns and bridges
- Framework for full veneers
- Model casting constructions
- Telescopic abutment constructions

7. Notes on Design

Minimum wall thickness:

- 0,6 mm (zervikal)
- 1,2 mm (okklusal)

8. Notes on milling and further processing

We recommend using a PMMA strategy in conjunction with single-cutting or wet milling. A fine-toothed carbide bur must be used to remove the milled frameworks from the milling blank. The restorations can be finished with suitable silicone polishers and polishing brushes.

High-gloss polishing is carried out with conventional polishing pastes.

9. Individualization

Compounding with cold-curing polymers and composites is possible. We recommend the use of our own products or cold-curing polymers from AcrylX

10. Cleaning

For gentle cleaning, we recommend conventional soapy water or detergent in an ultrasonic device or hand brush.

11. Attachment

For the attachment of restorations made out of **Eldy Plus®** all proven provisory and permanent cements are suitable.

12. Interactions

Interactions with other products commonly used in dentistry are not to be expected. If interactions cannot be ruled out in the mouth, the dental prosthesis must be removed

13. Storage

Dental Plus Blanks should be stored in a dry place at room temperature and protected from direct sunlight

14. Expiry date

See label on the packaging

15. First aid measures

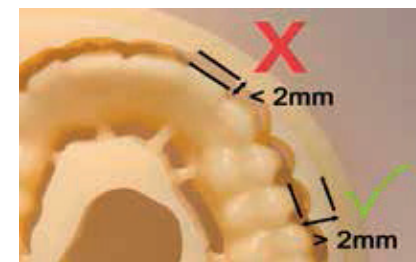
Due to the properties of the material, no damage to health is normally to be expected when processing **Dental Plus** Blanks. However, should any reaction occur due to special circumstances, please observe the following instructions:

- After inhalation: Seek medical advice if symptoms occur.
- After skin contact: Rinse with water and seek medical advice if symptoms occur.
- After eye contact: In case of mechanical irritation of the eyes, rinse with water and seek medical advice.
- After ingestion: If symptoms occur, seek medical advice.

16. Warnings

- **The use of denture finishers can lead to embrittlement (fracture) of the material if the corresponding ingredients are used.**
- Prolonged contact with high-proof alcohol solutions should be avoided. We recommend Bacoban for disinfecting plastics.
- Please note that improper use of milling tools and polishing brushes can cause the material to overheat. This can lead to damage to the material.

Please note when processing Dental Blanks **Eldy Plus Multicolor**:

0481